ARTHREX SHOULDER TRACTION AND ROTATION STaR™ SLEEVE ASSEMBLY INSTRUCTIONS

Directions for Use

ENGLISH

Shoulder Traction and Rotation
STaR™ Sleeve
Assembly Instructions

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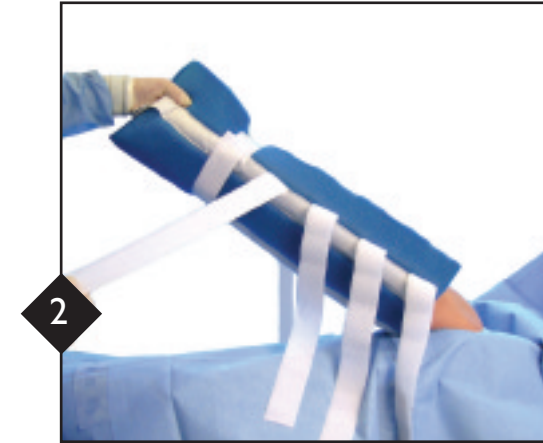
Measure the sleeve against the patient's arm so that the wrist and hand are positioned at the distal end of the sleeve. Trim the sleeve to extend no less than two inches inferior to the greater tuberosity.

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Either wrap the supplied Coban™ wrap around the sleeve distal-to-proximal or open the Velcro straps and wrap them around the sleeve distal-to-proximal.

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Attach the hook and cable (YELLOW) from the distal traction arm to the metal triangular ring on the end of the sleeve. Apply traction weights to the distal traction cable and position the traction arm for the optimum angle.

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ASSEMBLY INSTRUCTIONS CONTINUED...

Position the lateral traction and rotation control strap

4

Hang a sterile S-hook from the clip of the horizontal traction cable (RED). If used, hang the traction scale from the horizontal traction cable (RED) and place the other sterile S-hook on the distal end of the scale.

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5

Place the triangular ring of the rotation control strap on the S-hook so that the strap hangs down.

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6

Open and straighten the rotation control strap. Attach the blue end underneath the sleeve. Pass the white end under the opposite side of the arm and affix it to the rotation control strap. Secure the full length of the upper portion of the strap to itself.

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7

To control internal and external rotation during the procedure, loosen the Velcro strap, rotate the arm, and retighten the strap.

NOTE: Use the second sterile S-hook to maintain sterility if the rotation control strap is detached from the unsterile horizontal traction cable and needs to be reattached during the same surgical procedure.

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© Copyright Arthrex Inc., 2005
All rights reserved.
Arthrex, Inc.
Arthrex GmbH