## English

### A. DEVICE DESCRIPTION

The DynaNite® PIP (Proximal interphalangeal) implant is an implant with a barbed end, and a threaded end. Sizes range from 12-16 mm in overall length. The DynaNite PIP implant is inserted between the proximal and middle phalanges, so that the threads fixate one phalangeal canal and the barbed end fixates the other phalangeal canal of the toe.

- DynaNite PIP implants are designed to exhibit super-elastic properties.
- The implants are sold in two configurations:
  - As a part of a single use disposable kit. The kit includes an implant loaded into an inserter, drill bits, guide wire and a tamp
  - As an implant loaded into an inserter only

### B. INDICATIONS

The device is indicated to stabilize and aid in the fixation of fractures, fusions, and osteotomies of the phalanges.

### C. CONTRAINDICATIONS

- Insufficient quantity or quality of bone.
- Blood supply limitations and previous infections, which may retard healing.
- Foreign body sensitivity. Where material sensitivity is suspected, appropriate tests should be made and sensitivity ruled out prior to implantation.
- Any active infection or blood supply limitations.
- Conditions that tend to limit the patient’s ability or willingness to restrict activities or follow directions during the healing period.
- The use of this device may not be suitable for patients with insufficient or immature bone. The physician should carefully assess bone quality before performing orthopedic surgery on patients who are skeletally immature. The use of this medical device and the placement of hardware or implants must not bridge, disturb or disrupt the growth plate.
- Do not use for surgeries other than those indicated.
- Comminuted bone surface that would not allow for implant placement.
- Pathologic bone conditions such as osteopenia that would impair the ability to securely fix the implant.
- Foreign body sensitivity to metals including nickel. When material sensitivity is suspected, appropriate tests should be made prior to implantation.

### D. ADVERSE EFFECTS

- Infections, both deep and superficial.
- Foreign body reactions.

### E. WARNINGS

- An internal fixation device must never be re-used.
- Do not re-sterilize this device.
- Postoperatively and until healing is complete, fixation provided by this device should be considered as temporary and may not withstand weight bearing or other unsupported stress. The fixation provided by this device should be protected. The postoperative regimen prescribed by the physician should be strictly followed to avoid adverse stresses applied to the device.
- Pre-operative and operating procedures, including knowledge of surgical techniques and proper selection and placement of the device, are important considerations in the successful utilization of this device. The appropriate Arthrex delivery system is required for proper implantation of the device.
- Any decision to remove the device should take into consideration the potential risk to the patient of a second surgical procedure. Device removal should be followed by adequate postoperative management.
- Detailed instructions on the use and limitations of this device should be given to the patient.
- This is a single use device. Reuse of this device could result in failure of the device to perform as intended and could cause harm to the patient and/or user.
- Removal of supplemental fixation after healing. If the supplemental fixation is not removed following the completion of its intended use, any of the following complications may occur: (1) Corrosion, with localized tissue reaction or pain; (2) Migration of implant position resulting in injury; (3) Risk of additional injury from postoperative trauma; (4) Bending, loosening, and/or breakage, which could make removal impractical or difficult; (5) Pain, discomfort, or abnormal sensations due to the presence of the device; (6) Possible increased

risk of infection; and (7) Bone loss due to stress shielding. The surgeon should carefully weigh the risks versus benefits when deciding whether to remove the implant. Implant removal should be followed by adequate postoperative management to avoid re-fracture.

### F. MRI SAFETY INFORMATION

#### I. MR CONDITIONAL

*Non-clinical testing and in-vivo electromagnetic simulations demonstrated that the DynaNite PIP Implant is MR Conditional. A patient with this device can be scanned safely in an MR system immediately after placement under the following conditions:*

- Static magnetic field of 1.5-Tesla and 3-Tesla, only
- Maximum spatial gradient magnetic field of 3000 Gauss/cm or less
- Maximum MR system reported, whole body averaged specific absorption rate (SAR) of 2-W/kg for 15 minutes of scanning in the Normal Operating Mode of operation for the MR system
- Under the scan conditions defined, the DynaNite PIP Implant is expected to produce a maximum temperature rise of 5.2°C after 15-minutes of continuous scanning.

#### a. Artifact Information

*In non-clinical testing, the image artifact caused by the DynaNite PIP Implant extends approximately 66 mm from this implant when imaged using a gradient echo pulse sequence and a 3-Tesla MR system.*

### G. PRECAUTIONS

- Surgeons are advised to review the product-specific surgical technique prior to performing any surgery. Arthrex provides detailed surgical techniques in print, video, and electronic formats. The Arthrex website also provides detailed surgical technique information and demonstrations. Or, contact your Arthrex representative for an onsite demonstration.

### H. PACKAGING AND LABELING

- Arthrex devices should be accepted only if the factory packaging and labeling arrive intact.
- Contact Customer Service if the package has been opened or altered.
- All of the symbols used on the labeling along with the title, description and standard designation number may be found on our website at [www.arthrex.com/symbolsglossary](http://www.arthrex.com/symbolsglossary).

### I. STERILIZATION

This device is provided sterile. Refer to the package label for the sterilization method.

Certain Arthrex instruments that may be used during this procedure are provided non-sterile and must be adequately cleaned and sterilized prior to use or re-use. Please refer to DFU-0023-XX and ANSI/AAMI ST79, “Comprehensive Guide to Steam Sterilization and Sterility Assurance in Health Care Facilities” for specific information.

### J. MATERIAL SPECIFICATIONS

Refer to the package label for the materials. This device is made of Nitinol.

### K. STORAGE CONDITIONS

Sterile devices must be stored in the original unopened packaging, away from moisture and should not be used after the expiration date.

### L. INFORMATION

Surgeons are advised to review the product specific surgical technique prior to performing any surgery. Arthrex provides detailed surgical techniques in print, video, and electronic formats. The Arthrex website also provides detailed surgical technique information and demonstrations. Or, contact your Arthrex representative for an onsite demonstration.

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## Arthrex, Inc.
## Arthrex GmbH