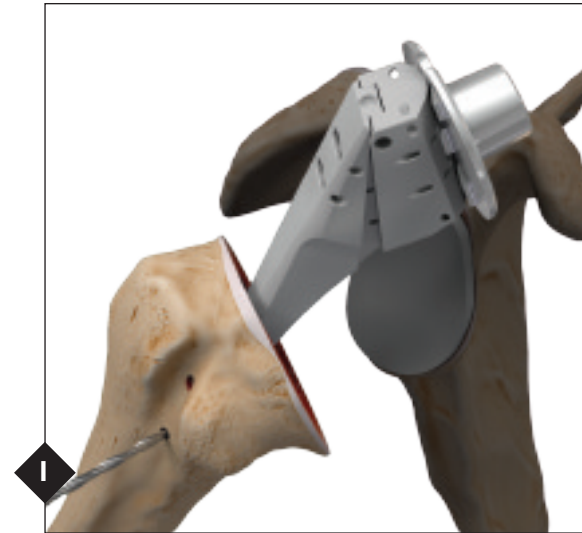
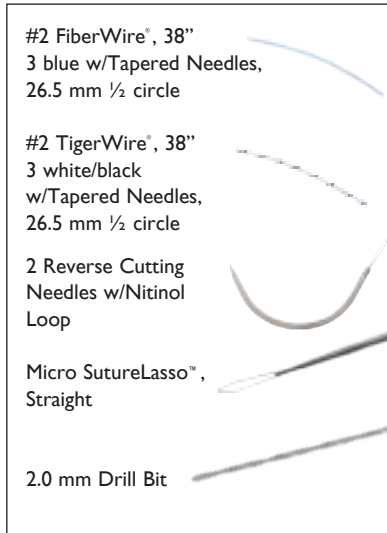
**UNIVERS™ APEX SUTURE KIT**  
Subscapularis Repair Technique

**ENGLISH**

Two holes are drilled (2 mm drill bit recommended) vertically in the biceps groove for passage of the lateral suture limbs. Holes should line up with the sutures in the prosthesis. The Micro SutureLasso (AR-8703) aides the passage of lateral sutures through the drill holes.

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
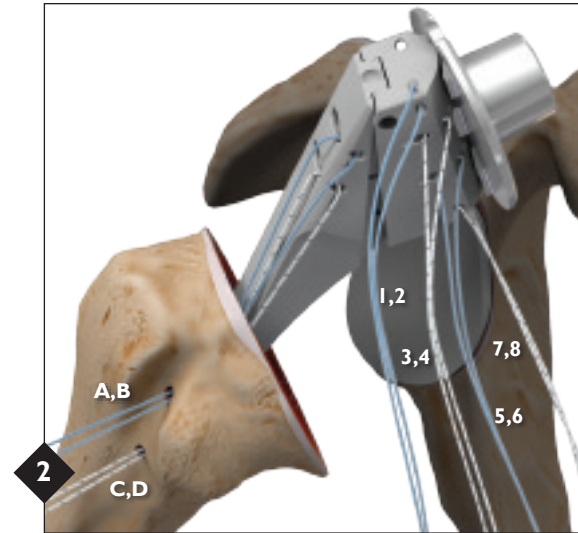
Load sutures in the stem from superior to inferior. Suture colors are:  
Lateral eyelets - blue, white/black  
Medial eyelets - blue, white/black, blue, white/black

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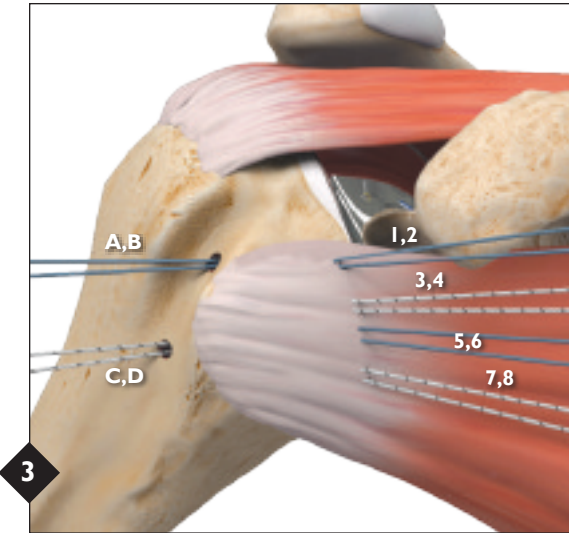
The eight (8) medial suture limbs are passed through the medial aspect of the subscapularis tendon. Sutures should be evenly spaced.

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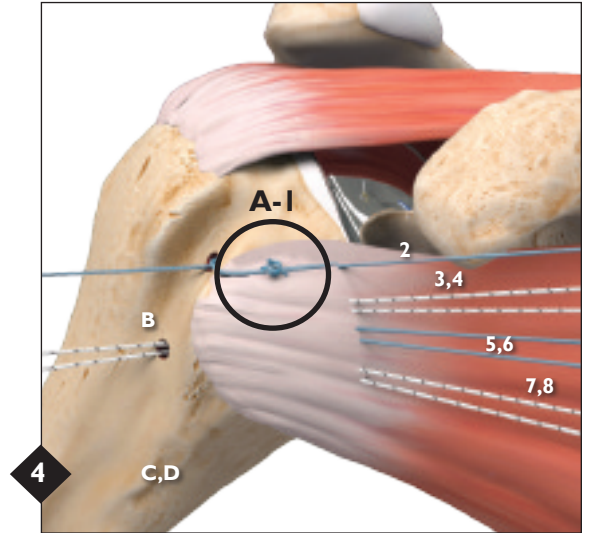**Knot I: A to I**  
From superior to inferior: lateral sutures are labeled A-D, medial sutures are labeled 1-8.

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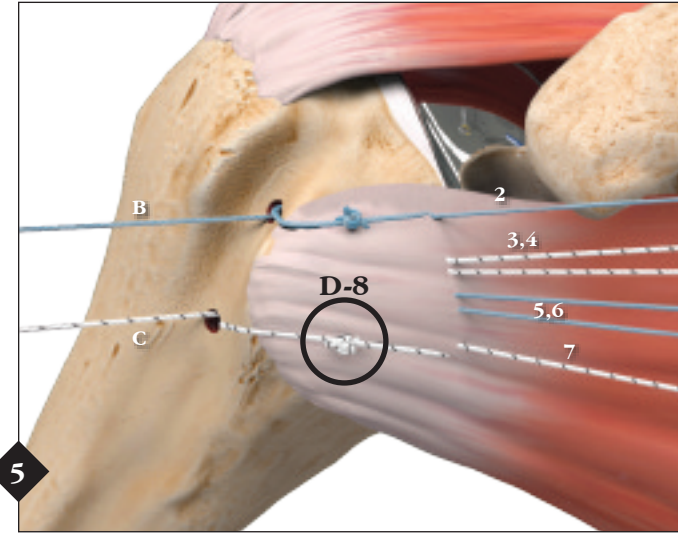
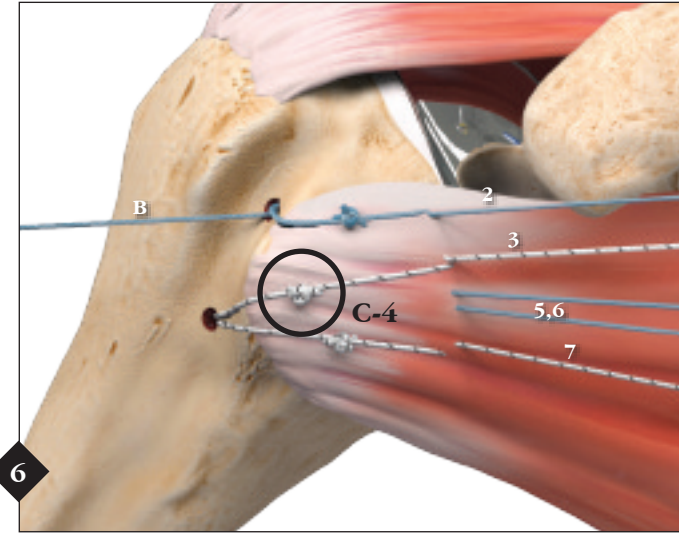
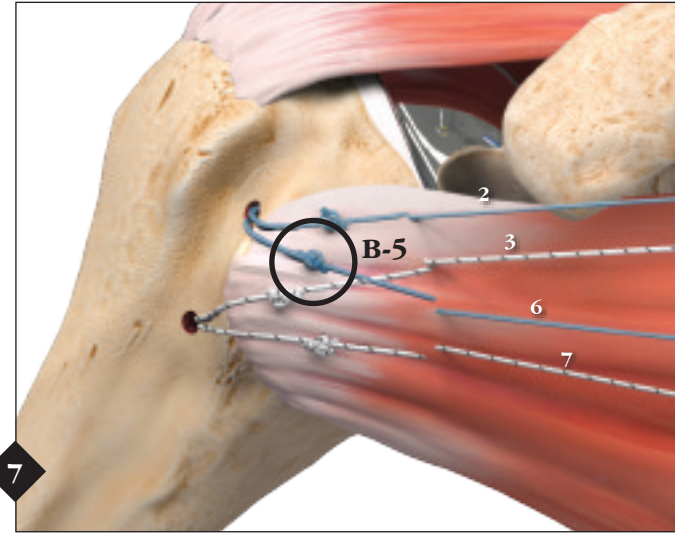
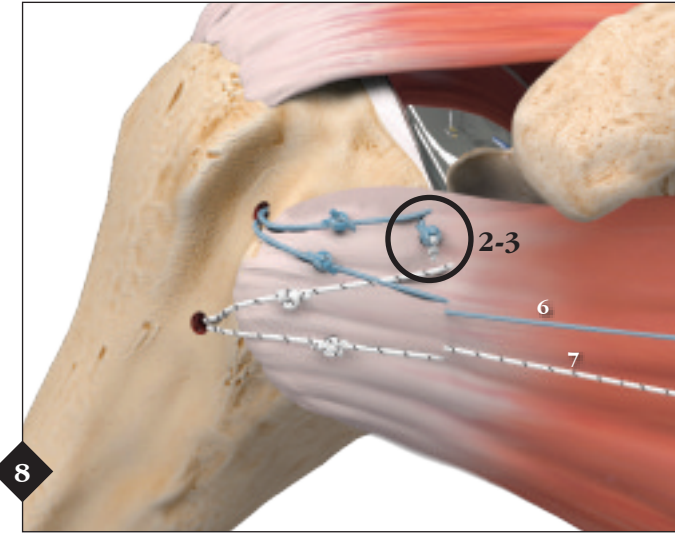
**Nudo 4: B a 5**

**Knot 5: 2 to 3**

Tension the sutures by direct pull to take slack out of the suture construct. Avoid a sawing motion as this can rupture the sutures.

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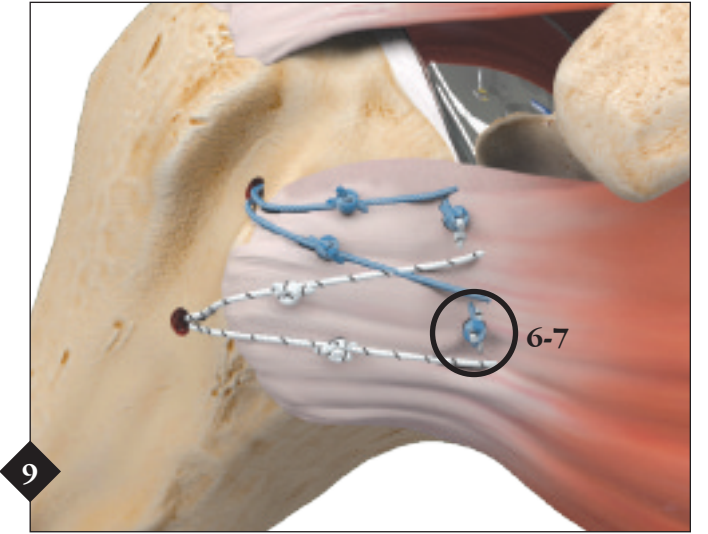
**Knot 6: 6 to 7**

Dynamic compression of the tendon occurs over the lesser tuberosity as the suture construct is tensioned. Consider additional sutures in the rotator interval to complete repair.

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