
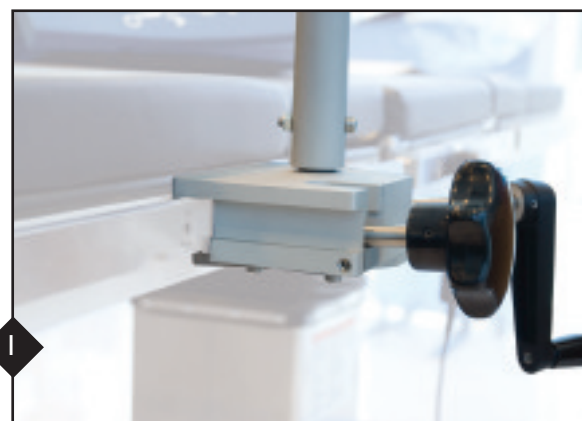
ARTHREX LATERAL DECUBITUS TRACTION TOWER ASSEMBLY INSTRUCTIONS (AR-1630)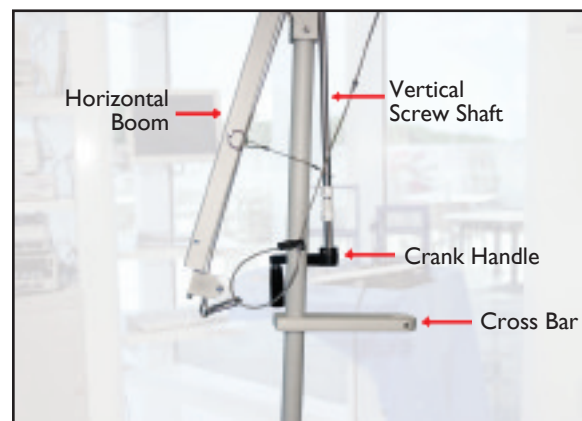

ENGLISH

The photo above of the Arthrex Lateral Decubitus Traction Tower shows the key components mentioned in the following assembly instructions.

Attach the tower base to the Clark Rail close to the foot of the O.R. table on the side of the bed in which the patient will be facing.

Raise the horizontal boom by placing the groove of the plastic sleeve on the vertical screw shaft into the slot in the cross bar. The vertical screw shaft is then locked in place by inserting the quick release pin through the holes in the cross bar.

Use the crank handle on the base to rotate the horizontal boom to be in line with the patient's operative shoulder.
Note: The horizontal boom will automatically lock in place and can be adjusted intraoperatively by simply turning the crank handle clockwise or counterclockwise.

The vertical screw shaft crank handle is used to raise or lower the horizontal boom to adjust the abduction of the operative arm.

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ASSEMBLY INSTRUCTIONS CONTINUED...

The weight hanger rod is attached in the square hole of the adjustment clip on the end of the vertical cable. The weights should be positioned below the patient level of the O.R. table, but not touching the floor.

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When the traction tower is in the desired position and the weight hanger does come in contact with the floor, the cable may be shortened by placing the adjustment clip over one of the three adjustment balls swaged onto the cable. The weight hanger rod can be attached in the loop of the cable.

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After the patient's arm is secured in the STaR™ Sleeve, the "S" hook (supplied with the STaR Sleeve) is placed in the nonsterile spring clip attached to the end of the horizontal cable. Weights can be added to the weight hanger rod until the desired traction is achieved.

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Storage: With the horizontal boom parallel with the floor, remove the quick release pin from the cross bar and slide the vertical screw shaft out of the slot in the cross bar. Lower the horizontal boom to the vertical position. The quick release pin can now be inserted into the holes of the horizontal boom to secure it for transport. After removal from the Clark Rail, the traction tower can be mounted on the optional storage stand (AR-I600SS).

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Arthrex, Inc.
Arthrex GmbH