ARTHREX HIP DISTRACTOR SYSTEM ASSEMBLY INSTRUCTIONS

ENGLISH

Countertraction:

A sandbag may be taped to the O.R. table, positioned under the operative extremity, just distal to the patient's hip. No center post is needed (as used with a fracture table).

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The patient is prepped and draped in standard fashion.

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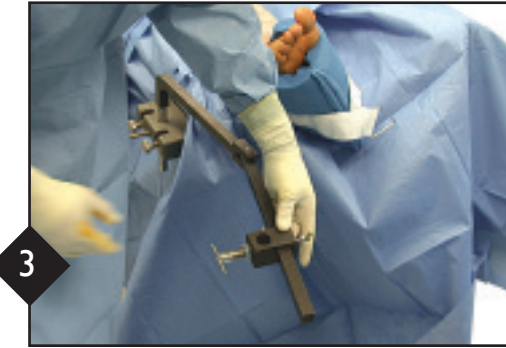
The sterile foot cuff is placed on the extremity and secured with Coban.

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
The Clark Rail Adapter is secured onto the side rail of the O.R. table just proximal to the foot. The long distractor arm is inserted into the adapter. Secure the joint of the arm with a T-screw.

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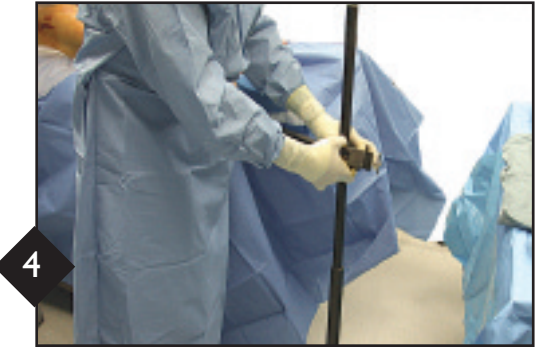The kickstand is inserted and secured into one distractor clamp at the end of the distractor arm with the distractor arm parallel to the extremity.

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ASSEMBLY INSTRUCTIONS CONTINUED...

5

Attach the traction pulley bar to the long distractor arm with a clamp near the kickstand so weights will hang freely off the pulley.

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6

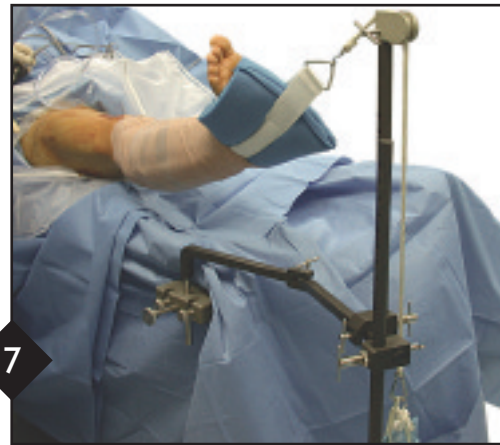
Attach the sterile rope and hook to the foot, run the rope over the pulley groove and down. Attach the desired amount of weight (40-50 lbs.).

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7

Final set-up and positioning of the Hip Distractor. Note 20° of hip flexion allowing for unobstructed traction to the extremity off the O.R. table.

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8

Confirm optimum joint distraction via C-arm. The foot may be easily externally/internally rotated.

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Materials: Velcro™, nylon, Coban™, aluminum and stainless steel

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Materialies: Velcro, nylon, Coban, aluminio y acero inoxidable

Disassemble prior to resterilization. Refer to DFU-0023 for sterilization information.

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Arthrex, Inc.
Arthrex GmbH