Scorpion[®] -multifire

Instructions for Loading Device

Arthrex, Inc.
1370 Creekside Boulevard,
Naples, Florida 34108-1945 USA
800-934-4404 • Fax: 800-643-9310
www.arthrex.com

Arthrex GmbH
Erwin-Hielscher-Strasse 9
81249 München, Germany
Tel: +49 89 909005-0
www.arthrex.de

DFU-0227, rev. 0

© 2016, Arthrex Inc. All rights reserved. PATENT PENDING.

WARNING:

DO NOT RESTERILIZE OR REUSE the Scorpion needle. This may cause the needle to break and cause patient injury.

Use of excessive force to pass the needle through fibrous/calcific tissue, or striking bone, can cause needle breakage and patient injury. To avoid this, reposition the needle pass to a different area.

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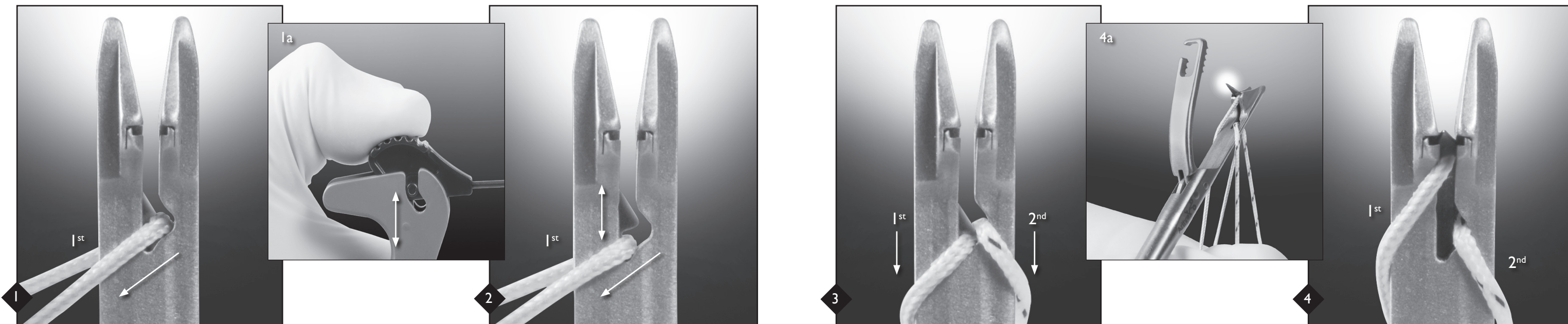
ENGLISH

1. Load 1-2 inches of the 1st #2 FiberWire[®] into the slot on the bottom jaw with its short, free end down. Grab both strands of FiberWire and pull them back and to the left side. Place your other hand on the back end of the device, with your thumb on top of the black thumbpad (1a).

2. Push down on the black thumbpad. The needle advances and the suture moves into the slot of the needle. Release the thumbpad. The needle will retract to its original position.

3. Load 1-2 inches of the 2nd #2 FiberWire strand into the slot with its short, free-end down. Grab all four strands of FiberWire and pull back with the 1st loaded suture strands on the left and the 2nd loaded strands on the right. Place your other hand on the handle of the device, avoiding the front lever.

4. Gently squeeze the handles with the palm of your hand until you see the tip of the needle (4a). Release the handles and the needle will retract. The device is loaded and passes the suture you loaded first. Fully squeeze the handles to reset the needle position and remove sutures, if needed.

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