PROWICK™ SHOULDER COLD THERAPY WRAP INSTRUCTIONS FOR USE (AR-1625P)

ENGLISH

Description/Indications for use:

The ProWick Cold Therapy Wrap is intended for providing cold therapy during the first 48 hours after arthroscopic or mini-open shoulder surgery. The ProWick system consists of one ProWick Cold Pack and one Elastic Compression Strap.

ENGLISH

CONTRAINDICATIONS/WARNINGS

None known at this time.

PRECAUTIONS

CAUTION: Federal law (USA) restricts this device to sale by or on the order of a physician.

- The ProWick Cold Therapy Wrap is not intended to provide heat therapy. Do not heat the Cold Pack in a microwave oven or by any other method.
- Do not allow ProWick Cold Therapy Wrap to get wet.
- Do not place Cold Pack directly against skin.
- Cold therapy should be applied per the hospital/physician protocol.

HOW SUPPLIED

The ProWick Cold Therapy Wrap is supplied sterile as part of the ProWick Shoulder Postoperative Dressing and Cold Therapy System.

STORAGE AND HANDLING

Store in a cool, dry place and in a manner that protects the integrity of the Cold Pack.

PREPARATION FOR USE

Read the Instructions For Use in their entirety prior to using the device. Inspect for damage to product or sterile barrier. Do not use if the product is damaged. The ProWick Cold Therapy Wrap is to be used after application of the ProWick Shoulder Postoperative Dressing.

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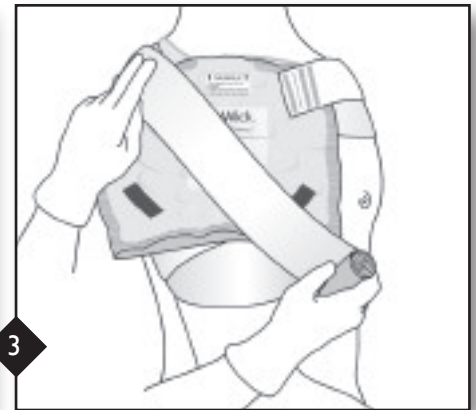
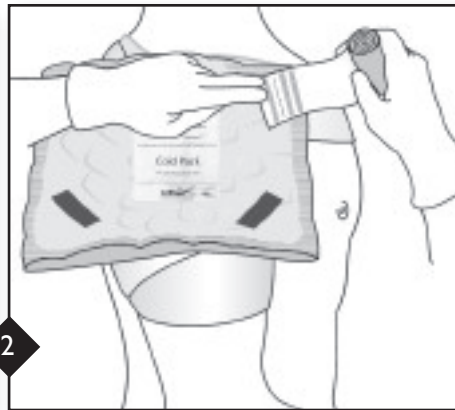
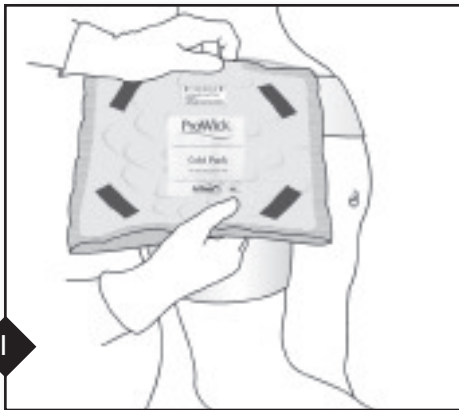
ENGLISH

Instructions for use:

1. Fill the ProWick Cold Pack with ice (cubed or crushed) to about 3/4 full and purge air. Tightly roll spout/opening down to bag level and snap straps across spout to secure. Place the Cold Pack over the ProWick Shoulder Postoperative Dressing so that the label is facing away from the patient.
2. With the hook strips and label on the Elastic Compression Strap facing away from the patient, attach the end of the Elastic Compression Strap to the hook strips on the Cold Pack located on the upper corner of the Cold Pack near the patient's chest.
3. Wrap the free end of the Elastic Compression Strap around the patient's torso and over the hook strips on the posterior side of the Cold Pack near the patient's head.
4. Continue wrapping the Elastic Compression Strap once around the biceps then back over the shoulder, attaching it to the exposed hook strips on the end of the Elastic Compression Strap already secured to the Cold Pack. If applicable, cut off excess length after securing the strap.

When the ice has melted, remove Elastic Compression Strap and Ice Pack by disengaging the hook strips. Empty the Cold Pack. Refill with ice per physician's instructions. Replace the ProWick Cold Pack and Elastic Compression Strap in the same manner as described in steps 1-4. Do not allow the ProWick Shoulder Postoperative Dressing to get wet. Do not remove the ProWick Shoulder Postoperative Dressing until instructed to do so by physician. After dressing removal, the patient may continue to use cold therapy per physician protocol. Patient should apply waterproof bandages per physician recommendations after dressing removal.

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Velcro is a registered trademark of Velcro Industries B.V.
© 2011, Arthrex Inc. All rights reserved. LA11624, rev. 3
Arthrex, Inc.
Arthrex GmbH