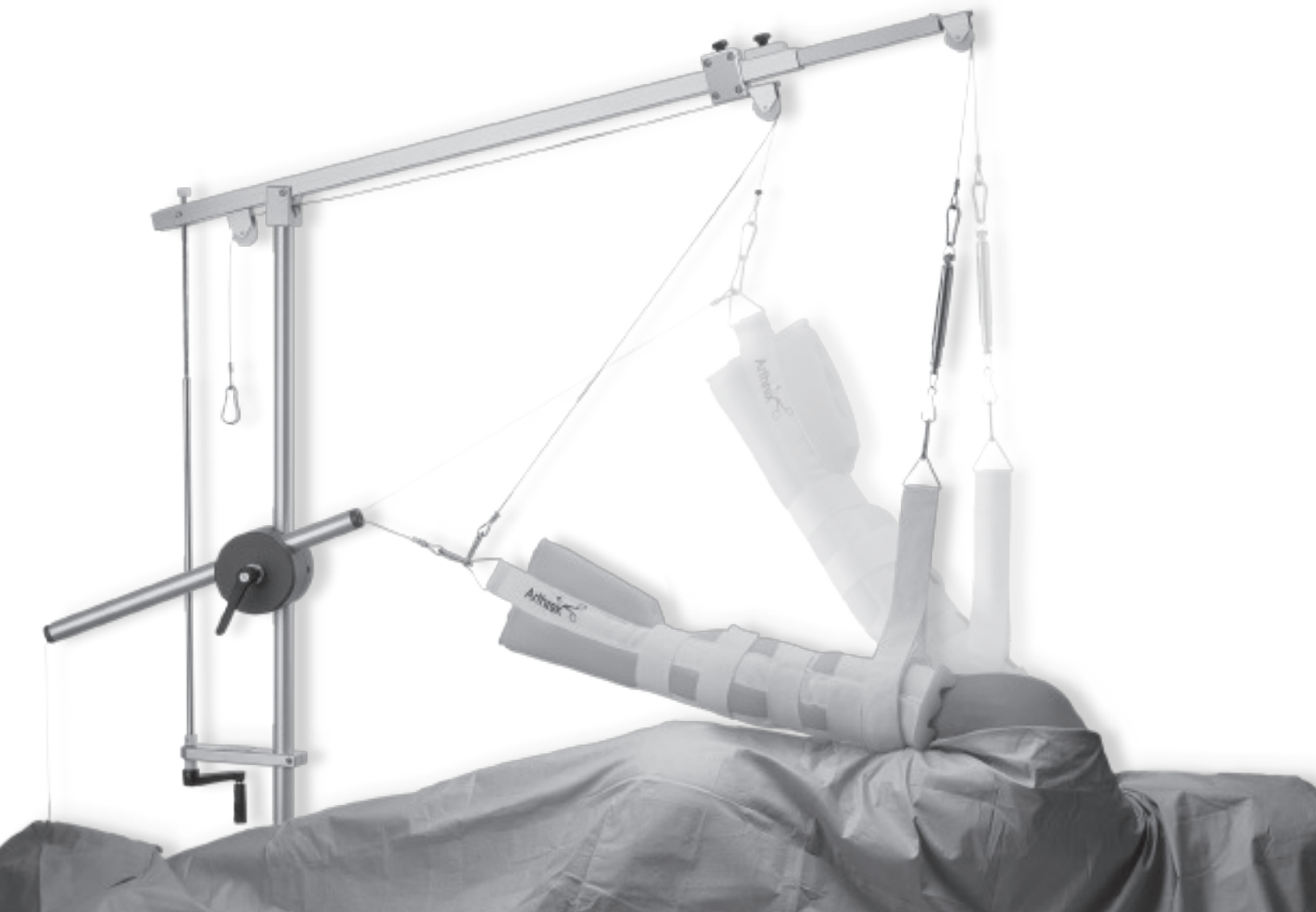
ARTHREX 3-POINT SHOULDER DISTRACTION SYSTEM (AR-1600M)

ASSEMBLY INSTRUCTIONS

ENGLISH

Unpack the assembly from the box and remove the protective wrapping. Remove the boom swivel adjustment knob (H) from the packaging and assemble to the large block near the bottom of the traction assembly. Unscrew and remove the extension arm elevation angle adjustment stop (E). Rotate the handle assembly (G) affixed to the screw shaft while aligning the toggle nut to start the thread. When started, turn the handle clockwise until at least an inch of thread is protruding through the top of the toggle nut. Reassemble the stop to the top of the shaft (E). This is important to prevent inadvertently backing the shaft completely out of the toggle nut. Adjust the boom orientation by sliding the bar holding the adjustment screw to the vertical tube, so that it is approximately 18 - 20 inches from the bottom of the device. Slide the base of the traction assembly onto the Clark Rail on the side of the table opposite the operative arm. Secure the shoulder holder to the rail by tightening the two small black rail attachment knobs (I). Adjust the boom orientation by rotating the handle (H) affixed to the screw shaft as needed. Attach standard traction weights as required. *Note: No routine maintenance is required. Thoroughly wipe all surfaces of the device with a mild cleaning solution or disinfectant and a damp cloth. Do not clean the device with abrasive cleaning or disinfectant compounds, solvents, or other materials that could scratch or damage the device.*

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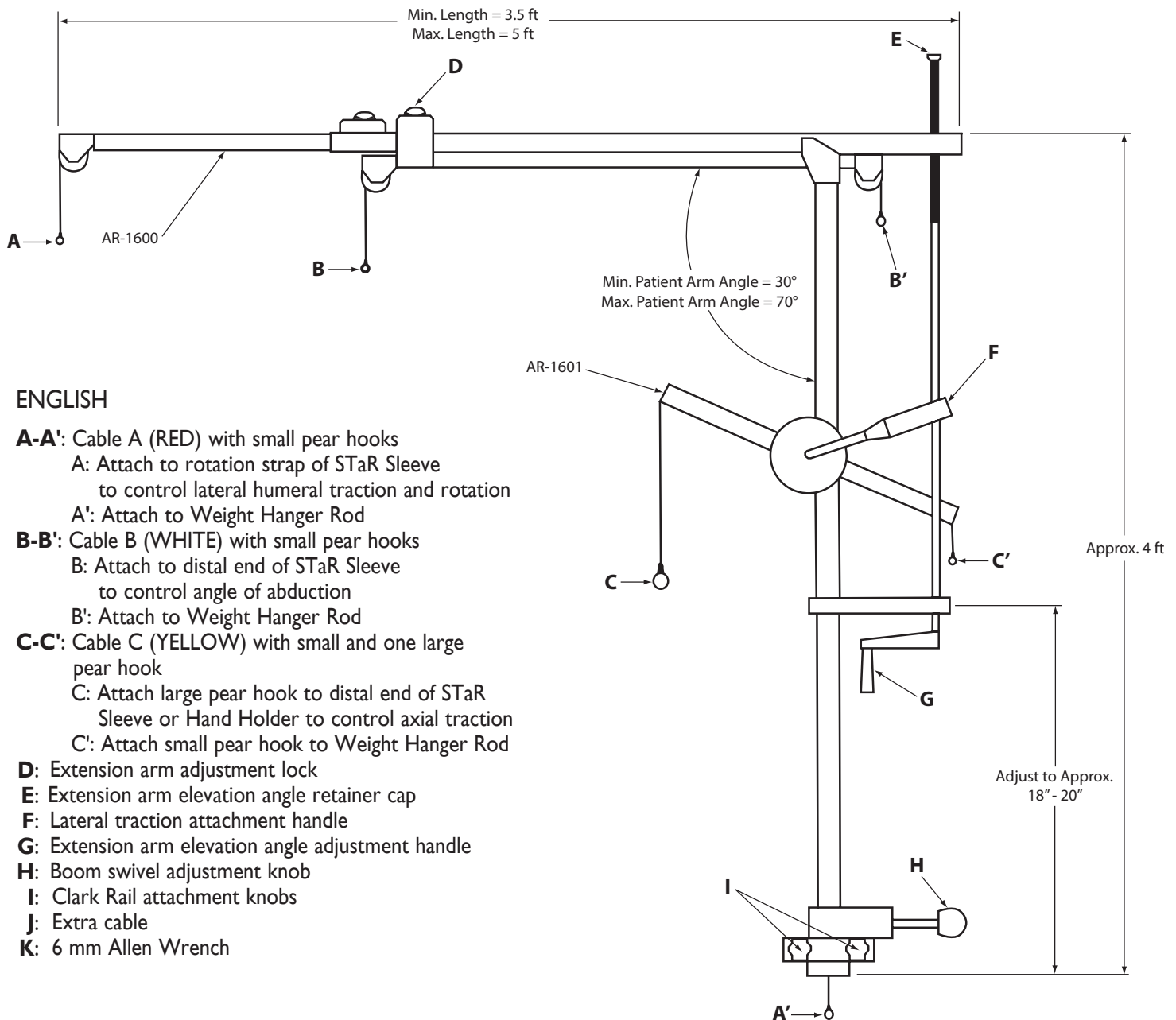
ENGLISH

- A-A'**: Cable A (RED) with small pear hooks
 - A: Attach to rotation strap of STaR Sleeve to control lateral humeral traction and rotation
 - A': Attach to Weight Hanger Rod
- B-B'**: Cable B (WHITE) with small pear hooks
 - B: Attach to distal end of STaR Sleeve to control angle of abduction
 - B': Attach to Weight Hanger Rod
- C-C'**: Cable C (YELLOW) with small and one large pear hook
 - C: Attach large pear hook to distal end of STaR Sleeve or Hand Holder to control axial traction
 - C': Attach small pear hook to Weight Hanger Rod
- D**: Extension arm adjustment lock
- E**: Extension arm elevation angle retainer cap
- F**: Lateral traction attachment handle
- G**: Extension arm elevation angle adjustment handle
- H**: Boom swivel adjustment knob
- I**: Clark Rail attachment knobs
- J**: Extra cable
- K**: 6 mm Allen Wrench

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Arthrex, Inc.
Arthrex GmbH