| |
|--|
| <p><i>Symbols Used on Labeling</i> Auf der Verpackung verwendete Symbole Symboles utilisés sur l'etiquetage Simboli usati sull'etichetta Símbolos utilizados en las etiquetas Símbolos utilizados na embalagem</p> |
|--|

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| English | DFU-0176r4 |
|--|--|
| A. INDICATIONS The Arthrex <i>Inform™</i> Low Fluid Alarm (LFA) is designed to monitor irrigant levels during arthroscopic procedures and inform operating room personnel when a low amount of fluid remains by weight. The device is intended for use with standard irrigation solutions such as saline in one, three, and five liter bags. <ul style="list-style-type: none">The device is NOT intended for use with intravenous infusion solutions or procedures. The device is NOT intended to be used with glass fluid containers. | Silent Mode: If the user wishes to turn off the auditory buzzer while device is in an alarm state (either yellow or red LED is illuminated), press and release the button. The LED's will continue to illuminate indicating a particular alarm state, but the buzzer will not sound. To bring device back to regular mode, press and release button again while device is in an alarm state. Device will be reset to regular mode once a new irrigant bag has been placed on the device. The five (5) second auditory alarm that indicates an empty bag can NOT be silenced. |
| B. CAUTIONS <ol style="list-style-type: none">Do NOT sterilize this device. Do NOT autoclave this device. NEVER immerse this device in water or cleaning detergent. This unit is NOT suitable for use in the presence of flammable mixtures. Keep this device away from magnetic and electromagnetic fields. Do NOT clean the device with abrasive cleaning or disinfectant compounds, solvents, or other materials that could scratch or damage the device. | Low Battery State: When the battery power reaches 25% remaining, the green LED will begin blinking at a fast rate. This gives the user a warning that the device will require battery charging soon. It will continue to blink at this quick rate until 5% of battery power remains. |
| C. PACKAGING AND LABELING <ol style="list-style-type: none">Arthrex devices should be accepted only if the factory packaging and labeling arrive intact. Contact Customer Service if the package has been opened or altered. | Battery Depleted State: Once the battery power has reached 5% remaining, the green LED will blink five (5) times and simultaneously give five (5) auditory beeps indicating a very low battery. After these five beeps the device will power off. |
| D. CLEANING AND DISINFECTION The <i>Inform™</i> Low Fluid Alarm is supplied non-sterile and is not to be sterilized. <p>To clean, unplug the power cord from the wall outlet and device. Thoroughly wipe all surfaces of the device and power cord with a mild cleaning solution or disinfectant and a damp cloth. Follow the procedures approved by your institution or use a validated infection control procedure.</p> <ul style="list-style-type: none">Do NOT clean the device with abrasive cleaning or disinfectant compounds, solvents, or other materials that could scratch or damage the device. Do NOT allow fluids to enter the device. | Charging the LFA: To charge the LFA, plug the power cord into the LFA and the other end into a powered outlet. The charging states are defined below. <p><i>Totally Charged:</i> Green LED is solidly illuminated <i>Charging:</i> Green LED is blinking slowly The power button is disabled while the device is plugged in; however the button may still be used to silence the device in the alarm state.</p> <p><u>NOTICE TO USER PLUGGING IN DEVICE FOR THE FIRST TIME OUT OF BOX:</u> It is required to charge device before using for the first time. Upon initially plugging in power cord, no LED's will be illuminated. It will take 10-15 minutes of charging for the green LED to appear with a slow blink, indicating device is receiving a charge.</p> |
| E. PERIODIC INSPECTION Every six months, visually inspect the Arthrex <i>Inform™</i> Low Fluid Alarm device for signs of wear or damage. <p>In particular, look for any of the following problems:</p> <ul style="list-style-type: none">Damage to the power cord Damage to the power cable receptacle Obvious damage to the unit Damage to any receptacle Accumulation of lint or debris in or around the unit | NOTICE TO USER PLUGGING IN DEVICE FOR THE FIRST TIME OUT OF BOX: It is required to charge device before using for the first time. Upon initially plugging in power cord, no LED's will be illuminated. It will take 10-15 minutes of charging for the green LED to appear with a slow blink, indicating device is receiving a charge. |
| F. STORAGE CONDITIONS There are no special storage requirements for this device. | |
| G. TECHNICAL SPECIFICATIONS <ul style="list-style-type: none">Power Supply: Emerson DA12-050US-M (U.S.) or Emerson DA12-050EU-M (EU) power supply and recharging plug. Only recharge the <i>Inform™</i> using one of these power supplies. Input: 100-240V~, 50-60Hz, 0.3A (max) Output: 5VDC, 2A (max), 10W(max) Internally powered by one (1) 3.7V Li-ion Rechargeable Battery Mode of operation is continuous. There are no applied parts for this device. | |
| H. DIRECTIONS FOR USE The LFA should be inspected for damage and appropriateness for use prior to each use. Hang the LFA on an irrigant fluid bag pole using its top hook. Turn on the power to the LFA by pressing and releasing the button. The green LED should illuminate indicating sufficient battery power. Attach a bag of irrigation solution to the lower hook. The yellow (amber) and red LED's should NOT be illuminated which indicates a sufficient amount of fluid in the bag. Device will continue to monitor irrigant levels. As the irrigant level decreases by weight to previously determined levels, the yellow and red LEDs will illuminate. These two LEDs are used to indicate the various states of the device. The device states are defined below. | |

| | | | | | |
|--|--|---|--|---|--|
| REF | Catalog Number Bestellnummer Référence catalogue Número de catálogo Número de catálogo |  | Manufacturer Hersteller Fabricant Produttore Fabricante Fabricante | LOT | Lot number Chargenbezeichnung No. de lot Numero di lotto Número de lote Número de lote |
|  | Electronic Waste Elektronischer Abfall Déchet électronique Rifiuti elettronici Desperdicio electrónico Resíduos electrónicos | SN | Serial Number Seriennummer Numéro de série Numero di serie Número de serie Número de série |  | Do not reuse Nicht wiederverwenden! Ne pas réutiliser Monouso No reutilizar Não reutilizar |
|  | Contains phthalates Enthält Phthalate Contient des phtalates Contiene ftalati Contiene ftalatos Contém ftalatos |  | Non sterile Nicht steril Non stérile Non sterile No estéril Não estéril |  | Manufacture Date Herstellungsdatum Date de fabrication Data di produzione Fecha de fabricación Data de fabrico |
|  | Storage Temperature Range Lagertemperatur La Portée de Température d'emmagasine Gamma Di Temperature Di Immagazzinaggio Temperaturas de Almacenamiento Intervalo de temperatura de conservaço |  | See instructions for use Bitte Gebrauchsanweisung beachten Lire attentivement la notice d'utilisation Leggere attentamente il foglio illustrativo Ver instrucciones de uso Consultar instruções de utilização | | |
|  | Use by - year & month Verwendbar bis Jahr und Monat À utiliser avant le (mois/année) Da usarsi entro anno e mese Cadauca - año y mes Utilizar por – ano e mês | QTY | Quantity Quantität Quantité Quantità Cantidad Quantidade | | |
| EC REP | Authorized Representative in the European Community Bevollmächtigter in der Europäischen Gemein-schaft Mandatnaire dans la Communauté européenne Mandatario nella Comunità Europea Representante autorizado en la Comunidad Europea Representante autorizado na União Europeia |  | Not to be used if package is damaged Nicht verwenden, wenn die Verpackung beschädigt ist Ne pas utiliser si l'emballage est endommagé Non usare se la confezione è danneggiata No utilizar si el paquete ha sufrido algún desperfecto Não deve ser utilizado se a embalagem estiver danificada | | |
| STERILE  | Sterile unless the package is damaged or open. Method of sterilization - EO Steril, solange die Verpackung ungeöffnet und unbeschädigt ist. Sterilisationsmethode - EO Produit stérile si l'emballage n'a pas été ouvert ou endommagé. Méthode de stérilisation - EO Il prodotto è sterile se la confezione non è aperta o danneggiata. Metodo di sterilizzazione - EO Esteril mientras el envase no sea abierto o dañado. Método de esterilización - EO Estéril a não ser que a embalagem esteja danificada ou aberta. Método de esterilização - EO | | | | |
| STERILE  | Sterile unless the package is damaged or open. Method of sterilization - gamma radiation Steril, solange die Verpackung ungeöffnet und unbeschädigt ist. Sterilisationsmethode - Bestrahlung Produit stérile si l'emballage n'a pas été ouvert ou endommagé. Méthode de stérilisation - irradiation Il prodotto è sterile se la confezione non è aperta o danneggiata. Metodo di sterilizzazione - Raggi Gamma Esteril mientras el envase no sea abierto o dañado. Método de esterilización - irradiación Estéril a não ser que a embalagem esteja danificada ou aberta. Método de esterilização – radiação gama | | | | |
| CE | The product meets the essential requirements of Medical Device Directive 93/42 EEC. Das Produkt entspricht den grundlegenden Anforderungen der Richtlinie des Rates über Medizinprodukte 93/42/EWG. Ce produit est conforme aux exigences de la directive sur les dispositifs médicaux CEE 93/42. Il prodotto è conforme ai requisiti essenziali della Direttiva CEE 93/42 sui Dispositivi Medici. Este producto cumple con las normas básicas de la Directiva de productos médicos. 93/42 CEE. O produto cumpre os requisitos essenciais da Directiva de Dispositivos Médicos 93/42 EEC. | | | | |

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