Symbols Used on Labeling

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Símbolos utilizados na embalagem

All symbols shown may not apply to this device. Please refer to the package labeling for utilized symbols.

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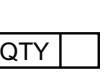
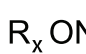
| | |
|---|--|
| English | DFU-0144r6 |
| A. DESCRIPTION The autoclavable Remote Control Units can operate the AR-6400 series Arthroscopy Pumps. The AR-6476 cannot be used with the AR-6480, Dual Wave Arthroscopy Pump. The AR-6480 uses the Dual Wave remote control (AR-6482) instead. The Remote Control Unit lets you adjust the settings from a maximum distance of 9.8 feet (3 meters) from the control unit of the pump. Note: The Run/Stop and Flush functions are inoperable in the original Continuous Wave Pump and Continuous Wave Pump II. | G. INFORMATION Surgeons are advised to review the product specific surgical technique prior to performing any surgery. Arthrex provides detailed surgical techniques in print, video, and electronic formats. The Arthrex website also provides detailed surgical technique information and demonstrations. Or, contact your Arthrex representative for an onsite demonstration. |

| |
|---|
| B. WARNINGS 1. Do not disconnect the remote control by pulling on the cable. Remove the remote control plug by grasping the body of the connector. 2. NEVER use liquid to clean the connector contacts of the remote control connector. Remove dust regularly with dry compressed air. |
|---|

| |
|---|
| C. CLEANING AND DISINFECTION The Remote Control can be cleaned and disinfected using commercially available surface disinfectants. Always comply with the instructions issued by the manufacturers of the cleaners and disinfectants. |
|---|

| |
|---|
| D. PACKAGING AND LABELING 1. Arthrex devices should be accepted only if the factory packaging and labeling arrive intact. 2. Contact Customer Service if the package has been opened or altered. |
|---|

| |
|--|
| E. STERILIZATION This device is provided non-sterile and can be sterilized. It is designed to survive a minimum 300 cycles of steam sterilization in an autoclave. It must be adequately cleaned, then sterilized using one of the following sterilization parameters. This unit is NOT designed for submersion or chemical sterilization. Follow your country-specific guidelines, standards, and requirements. |
|--|

| STERILIZATION PARAMETERS: FOR THE USA ONLY: | | | |
|--|----------------------|---------------|------------------|
| | Exposure Temperature | Exposure Time | Drying Time |
| Gravity-Displacement Steam Sterilization Cycle | 121°C (250°F) | 30 Minutes | 15 to 30 Minutes |
| | 132°C (270°F) | 15 Minutes | 15 to 30 Minutes |
| | 135°C (275°F) | 10 Minutes | 30 Minutes |
| Pre-vacuum Cycle | 132°C (270°F) | 4 Minutes | 20 to 30 Minutes |
| | 135°C (275°F) | 3 Minutes | 16 Minutes |

| STERILIZATION PARAMETERS: FOR OUTSIDE THE USA ONLY: | | | |
|---|-------------------------------|---------------|------------------|
| | Exposure Temperature | Exposure Time | Drying Time |
| Gravity-Displacement Steam Sterilization Cycle | 132°C – 135°C (270°F – 275°F) | 18 Minutes | 15 to 30 Minutes |
| | 121°C (250°F) | 30 Minutes | 15 to 30 Minutes |
| Pre-vacuum Cycle | 132°C - 135°C (270°F - 275°F) | 4 Minutes | 20 to 30 Minutes |

Certain Arthrex instruments that may be used during this procedure are provided non-sterile and must be adequately cleaned and sterilized prior to use or re-use. Please refer to DFU-0023 and ANSI/AAMI ST79, “Comprehensive Guide to Steam Sterilization and Sterility Assurance in Health Care Facilities” for specific information.

Sterilizers vary in design and performance characteristics. Cycle parameters and the load configuration should always be verified against the sterilizer manufacturer's instructions.

Cooling – The device must be adequately cooled, after being removed from the sterilizer.

After autoclave sterilization, the device must be fully dry before use.

| |
|---|
| F. STORAGE CONDITIONS Products must be stored in the original unopened packaging, away from moisture. |
|---|

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