ENGLISH

CONTRAINDICATIONS/WARNINGS

None known at this time.

PRECAUTIONS

CAUTION: Federal law (USA) restricts this device to sale by or on the order of a physician.

- The ProWick Knee Dressing should be removed within 48 hours after placement.
- Do not immerse dressing in fluid or expose to excess moisture, as dressing could become saturated.
- ProWick Knee Postoperative Dressing System is for SINGLE USE ONLY. DO NOT RSTERILIZE OR REUSE.

HOW SUPPLIED

The ProWick Knee Postoperative Dressing and Cold Therapy System is supplied sterile and is for single use only. Do not resterilize.

STORAGE AND HANDLING

Store in a cool, dry place and in a manner that protects the integrity of the package and sterile barrier.

PREPARATION FOR USE

Read the Instructions For Use in its entirety prior to using the device. Inspect for damage to product or sterile barrier. Do not use if the product is damaged or the sterile barrier is compromised.

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ENGLISH

Materials:

- Polyethylene
- Polipropylene
- Acrylate adhesive
- Polyurethane

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No Contiene Látex

PROWICK™ KNEE POSTOPERATIVE DRESSING AND COLD THERAPY SYSTEM INSTRUCTIONS FOR USE (AR-1626P)

ENGLISH

Description/Indications for use:

The ProWick Cold Wrap Knee Dressing System contains an absorbent dressing for use during the first 48 hours after arthroscopic knee surgery. The single size dressing is designed to accommodate most knees. The ProWick Knee Dressing is intended for absorbing small amounts of body fluids that may exude from arthroscopic incisions after surgery. The ProWick Cold Wrap Knee Dressing System contains one Cover Dressing, two sets of five Foam Islands, three Compression Straps, one Cold Pack and five waterproof bandages.

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


Developed with

U.S. PATENT NO. 6,346,654 B1; and PATENTS PENDING
Velcro is a registered trademark of Velcro Industries B.V.

© 2012, Arthrex Inc., All rights reserved. LAI1626, rev. 2

| | |
|--|--|
| | |
| Arthrex, Inc. 1370 Creekside Boulevard, Naples, Florida 34108-1945 USA 800-934-4404 • Fax: 800-643-9310 www.arthrex.com | |
| Arthrex GmbH Erwin-Hielscher-Strasse 9 81249 München, Germany Tel: +49 89 909005-0 www.arthrex.de | |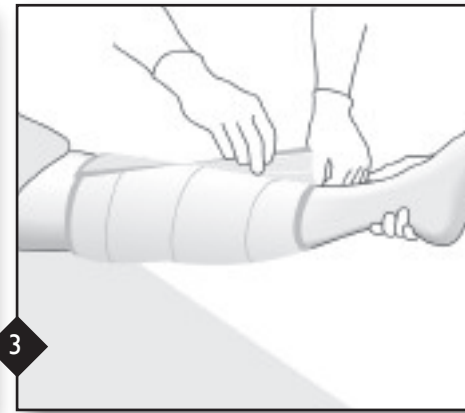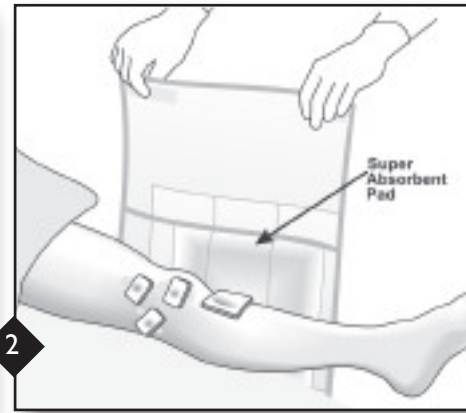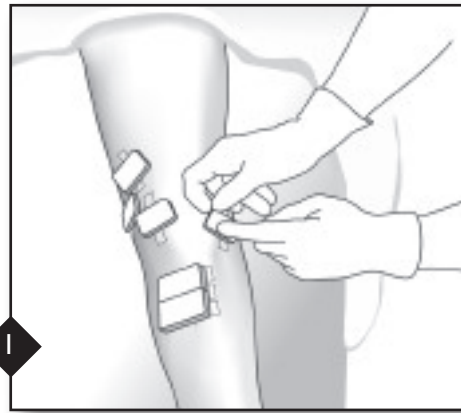

ENGLISH

Instructions for use:

When the operation is complete and the arthroscopic incision sites have been closed and dried per standard hospital procedure (typically suture and/or a SteriStrip™), the ProWick Knee Dressing System can be applied.

1. Peel a rectangular Foam Island off the release liner. (If desired and per hospital procedure, topical medication to relieve post-operative pain may be placed in the center of the Foam Island prior to placing the island over the wound site.) Apply the Foam Islands parallel to the incisions, and ensure that each incision site is completely covered by a Foam Island and that the adhesive is flush to the skin on all sides (Figure 1). For longer incisions, use two Foam Islands side-by-side to ensure the wound is completely covered.

2. Place the ProWick Knee Dressing (label facing out) so that all of the Foam Islands are fully covered by the dressing and the label is centered over the kneecap (Figure 2).

3. Wrap the dressing around the leg and secure it using the Velcro-like Hook Strips on the leading edge of the dressing (Figure 3).

4 & 5. Unroll one of the two large Compression Straps and attach it to the dressing at the calf by securing the Hook Strips to the outside of the dressing. Wrap the strap up and around the leg, in a candy cane stripe fashion (Figure 4). Secure the strap above the kneecap by pressing the Hook Strips to the dressing (Figure 5).

6. Secure a second Compression Strap to the top of the dressing (above the knee) using the Hook Strips. Wrap the strap down and around the leg, (moving toward the calf) in a

candy cane stripe fashion (Figure 6). Secure the end of the Compression Strap at the lower calf by pressing the Hook Strips to the dressing.

7. Adjust the Compression Straps as needed so that the fit is fairly snug, as tension will release as the surgical fluids are absorbed and the swelling decreases.

8. The ProWick Knee Dressing should be removed within 48 hours after placement. See Cold Pack and Waterproof Bandage Instructions For Use. The Instructions For Use should accompany the respective products to the location where they will be applied.

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