Sterile, Single Use Disposable Electrosurgical Pencils

Pencils are available in Push Button, Rocker, or Foot switching configurations. Each pencil is supplied with a coated electrode. Electrosurgical pencils are available with or without a holster.

This is a sterile, single-use product and is not designed for re-sterilization or reprocessing. Reuse and reprocessing pose risks of patient or user infection, patient injury, and may lead to device failure.

For use with maximum peak voltage of 4600 V.

Indication For Use

Disposable Electrosurgical Pencils are intended to remove tissue and control bleeding by use of high-frequency electrical current. The coated electrodes are intended as alternative to uncoated stainless steel electrodes or non-stick coated electrodes for use in conventional monopolar electrosurgical accessories. The coated electrodes are intended for use in situations where monopolar electrosurgical cutting and coagulation are normally used.

PRECAUTIONS AND WARNINGS:

GENERAL:

- Completely inspect package prior to use. Do not use if the sterile packaging has been opened or damaged.
- Completely inspect the product, especially the cable, before use for possible damage. Do not use electrode if any nicks or cuts are present on the insulation.
- Securely attach all components prior to use, and check the connection between the electrode and the accessory to ensure proper fit and compatibility. Improper electrode installation may result in injury to the patient or operating room personnel by arcing at the connection.
- Consult electrosurgical generator and dispersive electrode manuals to confirm proper electrosurgical generator power settings before proceeding with surgery. Use the lowest possible setting to achieve the desired surgical effect. (This is important due to the potential of inadvertent burning at high voltages.) Activate cut rather than coag since cut has a lower voltage.
- If the instrument is monopolar, a dispersive electrode should be used with the generator to prevent burns/injury to the patient.
- Aspirate fluids (e.g. blood or saline) from the area before activating the instrument to prevent unintended burns to the patient.
- Active the electrode only when you are ready to deliver electrosurgical current and the active tip is in view. Complete understanding of the principles of monopolar electrosurgery is necessary to avoid patient injuries. Deactivate electrode before the tip leaves the surgical site.
- Patient leads should be positioned in such a way that contact with the patient or other leads is avoided.
- Federal law (USA) restricts the sale of this device to the use or at the direction of a physician.
- This product is not manufactured with natural rubber latex.
- Do not arc the hemostat. Firmly grasp as much of the hemostat as possible before activating the generator. This disperses the current over a larger area, minimizing the current concentration at the finger tips.
- “Buzz the hemostat” below hand level (as close as possible to the patient) to reduce the opportunity for the current to follow alternate paths.
- When using a coated or nonstick electrode, place the edge of the electrode against the hemostat or other metal instrument.

FIRE HAZARD:

- Electrosurgical accessories that are activated or hot from use can cause a fire. Do not place them near or in contact with flammable materials. Place active accessories in a clean, dry, non-conductive holster when not in use, and keep away from patients and personnel to prevent burns.
- Both oxygen (O₂) and nitrous oxide (N₂O) are flammable. Do not use this product in oxygen- or nitrous oxide- enriched environments, especially during head and neck surgery, as this may cause fires or burns to patients or surgical personnel.

Assembly Instructions

1. Open package and place contents on sterile field.
2. Securely attach the electrode to the electrosurgical pencil.
3. Refer to the generator instruction manual for proper use.

Notice

1. Before inserting or changing an electrode, be sure that the active accessory is not connected to an electrosurgical generator or make sure that the generator is off (standby).
2. Grasp the electrode by the insulating sleeve, and insert the round shank into the electrosurgical accessory until the insulating sleeve is fully inserted. The shank and the insulating sleeve should fit securely into the active accessory. If the shank and/or insulating sleeve does not fit, or the insulation will not insert, use of the pencil is not recommended.
3. The electrode has a coating to reduce sticking of eschar. Cleaning the electrode with an abrasive pad may damage the electrode. If the coating or electrode is damaged, discard it.

SPECIAL NOTICE for NEEDLE ELECTRODE

1. Needle electrodes are fragile. Handle them with care to avoid damage to the needle and injury to hospital personnel.
2. Needle electrodes are designed for precise low power use during surgery. Using a needle at high power setting for long periods of time may result in damage to the needle.
3. Do not bend the needle electrode.

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