<p><i>Symbols Used on Labeling</i> Auf der Verpackung verwendete Symbole Symboles utilisés sur l'étiquetage Simboli usati sull'etichetta Símbolos utilizados en las etiquetas Símbolos utilizados na embalagem</p>
--

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English **DFU-0199r1**

ARTHREX saw blades made of high-grade stainless steel are optimally suited for their specific surgical use. They are designed, constructed and produced with utmost care. These quality instruments assure best working results and long service life provided they are used in the proper manner. Therefore, the following instructions for use and safety recommendations have to be observed. **Improper use may damage tissue and may lead to premature wear, destruction of the instruments and injury to the operator, patient or other persons.**

A. PROPER USE

- All saw blades are to be used in the appropriate handpieces/power systems only.
- Before use, make absolutely sure that the saw blades to be used and their packaging are in sterile and in technically sound condition. Discard any damaged saw blades and sterile packaging.
- Reciprocating saw blades should be inserted as deeply as possible. Saw blades which are not inserted to their full chucking length cause high centrifugal forces and vibrations.
- Check instruments for their perfect fit prior to use.
- The saw blades are not suitable for work on metal (e.g. implants).
- The saw blades should be actuated after insertion in the saw guide and before contact is made with the bone.
- When used in combination with templates, the cutting thickness of the saw blade should not exceed the thickness of the template. Ensure the blade moves freely in the template before it is used on the patient.
- Jamming, leverage and bending of the saw blade must be avoided while in the template. Otherwise the saw blade or template will be overheated which leads to jamming of the saw blade or thermal necrosis, or even the risk of fracture.
- Contact of the saw blade with the template, guiding device or other metallic objects has to be avoided. Otherwise, damage to the saw blade, template and guiding device can occur as well as blunting of the blades in addition to injury to the bone and surrounding tissue.

Improper use leads to unacceptable results and increased risk of injury.

B. INDICATION

ARTHREX saw blades have been developed for working on bone in the field of orthopaedics. The operator is responsible for choosing the appropriate saw blade according to the intended use.

C. CONTACT PRESSURE

Excessive contact pressure has to be avoided because this might lead to

- damage to the blade surfaces
- chipping of the blades
- reduced service life of saw blades
- excessive heat generation
- thermal necrosis

Excessive contact pressure may lead to overheating which in turn results in thermal necrosis; nicks in the blades may cause undesirable rough surfaces. At the worst, the instrument may break.

D. COOLING

Sufficient cooling should be provided when using saw blades. In case of insufficient cooling, the saw blade teeth will get clogged with bone chips, leading to increased heat generation. At worst, this may cause irreversible damage to the bone (thermal necrosis) and reduce service life of the saw blades considerably.

E. SAFETY AND LIABILITY

- The user is responsible for checking the product prior to use to ensure that it is suitable for the intended purpose.
- The user is responsible for the use of the instruments.
- In case of contributory negligence by the user, ARTHREX partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.
- **Do not regrind saw blades.**

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