A. DEVICE DESCRIPTION
The Interference Screw family includes the Soft Screw™, Interference Screw, Tenodesis Screw, RetroScrew™, Round Delta Interference Screw, Delta Tapered Interference Screw, FastThread™ Interference Screw and Bio-Cortical™ Screws.

B. INDICATIONS
Interference Screws, RetroScrews, and Bio-Cortical Screws. These devices are used for fixation of ligaments including ligament or tendon to bone, or a bone-to-bone joint. See below for specific indications.
Tenodesis Screw: Intended to provide soft tissue reattachment (i.e. fixation of ligament and tendon graft tissue).

C. CONTRAINDICATIONS
1. Bio-Tenodesis only: Screw sizes that are smaller than 7mm may not be appropriate for the local indication.
2. Insufficient quantity or quality of bone.

D. ADVERSE EFFECTS
1. Infections, both deep and superficial.
2. Foreign body reaction.
3. Allergic-like reactions to PLA materials (PLLA, PLLDA) have been reported.

E. WARNINGS
1. An internal fixation device must never be reused.
2. Metal implants only: All metallic implant devices used for this surgical procedure should be made of titanium.

F. MRI SAFETY INFORMATION
1. MRI Conditional
Non-clinical testing and in-vitro electromagnetic simulations demonstrated that the Interference Screws are MRI Conditional.

G. PRECAUTIONS
1. Surgeons are advised to review the product-specific surgical technique prior to performing any surgery.
2. Migration of the device from the implant position resulting in injury.

Table with 4 columns: Phase, Recirculation Time, Temperature, Detergent. Rows include Pre-Wash, Cleaning Wash, Neutralization Rise (optional), Thermal Disinfection, and Drying.

H. PACKAGING AND LABELING
1. Arthrex devices should be accepted only if the factory packaging and labeling are intact.
2. Contact Customer Service if package has been opened or altered.

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Table with 6 columns: Cycle Type, Minimum Exposure Temperature, Minimum Exposure Time, Minimum Drying Time, Minimum Cooling Time. Rows include US Pre-wash Cycle, US Pre-wash Cycle, and Pre-wash Cycle (Prion Cycle).

M. STORAGE CONDITIONS
Bioabsorbable devices must be stored in the original unopened packaging.
N. INFORMATION
Surgeons are advised to review the product specific surgical technique prior to performing any surgery.

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Arthrex GmbH
Arthrex, Inc.